



Oxivir® 1 Wipes

One-step Ready-to-use Hospital Disinfectant Cleaner Wipe

Next generation ready-to-use, pre-wetted, one-step disinfectant cleaner wipe, based on proprietary AHP® technology. Oxivir® 1 Wipes are formulated to deliver broad spectrum disinfection in ONE minute or less, with 75 total claims.

Features & Benefits

- Broad spectrum, multi-surface cleaner/disinfectant and sanitizer. Harnesses the cleaning power of hydrogen peroxide. Specially designed to penetrate the toughest soils
- Fast, Effective, Responsible and Sustainable formulation. Environmentally-responsible with no VOC's, no added fragrance, no NPE's. Fast-acting, convenient formula, suitable for daily use, and contact time of ONE minute or less
- Formulated with patented AHP® technology containing no phosphates, quats or phenols. The active ingredient breaks down to water and oxygen after use
- Short contact times of 1 minute or less Virucidal, 1 minute Bactericidal, Tuberculocidal and Fungicidal and 10-second non-food contact sanitizing
- Meets the lowest EPA toxicity category in all 6 toxicity studies, making it a responsible choice for users, around patients and on most common healthcare surfaces. Cleans, disinfects and deodorizes in one labor-saving step

Applications

- This product can be used in multiple areas including operating rooms, long-term care facilities, medical facilities, animal treatment facilities, cruise ships, hotels, public buildings, public restrooms, recreational facilities, retail businesses and schools (colleges and universities), and is compatible with most hard, non-porous surfaces





Oxivir® 1 Wipes

One-step Ready-to-use Hospital Disinfectant Cleaner Wipe






Use Instructions

- Pre-clean heavily soiled areas
- Wipe hard, non-porous environmental surfaces with towelette
- Allow surface to remain wet for 1 minute. (For certain listed viruses, allow to remain wet for 30 seconds).
- Air Dry, wipe surfaces to dry and remove any residue, or rinse with potable water as necessary

Note: All food contact surfaces must be rinsed with potable water. Do not use on glassware, utensils, or dishes.

Oxivir® 1 is compatible with most common healthcare surfaces. Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility, or contact a Diversey representative.

Technical data	Oxivir® 1 Wipes
EPA Reg. No	70627-77
Color/Form	Colorless, wipe
pH	1.7-2.5
Scent	Characteristic of ingredients
Shelf Life	2 Years

Product	Pack size	Dilution	Product code	
Oxivir® 1 Wipes	12 x 60 count wipes 7 in. x 8 in. / 17.8 cm x 20.3 cm Canisters	Ready-to-use	100850922	
Oxivir® 1 Wipes	12 x 160 count wipes 6 in. x 7 in. / 15.24 cm x 17.78 cm Canisters	Ready-to-use	100850923	
Oxivir® 1 Wipes	12 x 60 count wipes 10 in. x 10 in. / 25.4 cm x 25.4 cm Canisters	Ready-to-use	100962573	
Oxivir® 1 Wipes	4 x 160 count wipes 11 in. x 12 in. / 27.9 cm x 30.5 cm Buckets	Ready-to-use	100850924	
Oxivir® 1 Wipes	4 x 160 count wipes 11 in. x 12 in. / 27.9 cm x 30.5 cm Refills	Ready-to-use	100850925	

Safe handling

Please make sure your employees read and understand the product label and Safety Data Sheet before using this product. The label contains directions for use; and both the label and SDS contain hazard warnings, precautionary statements and first aid procedures. SDS are available online at www.diversey.com or by calling 888.352.2249. Improper use or dilution may result in damage to surfaces and may result in health and physical hazards that match those of the concentrate. Please refer to the Diversey HazMat Library, only available through Internet Explorer, <http://naextranet.diversey.com/dot/>, for up-to-date shipping information.